

Physician choice,
patient comfort.



PRODUCTS FOR
PERCUTANEOUS DRAINAGE



www.cookmedical.com

Cook Medical offers an unparalleled focus on patient comfort and one of the widest assortments of drainage products available.

- In the pages of this catalog, you'll find the right choice of drainage catheter to suit each procedure, and the accessories to complement your preference.
- You'll find the technology that only a leader can provide, including Ultrathane® construction for patient comfort, hydrophilic coating for ease of insertion, radiopacity for precise placement, and a variety of locking mechanisms.
- And you'll find the product qualities you have come to expect, such as kink resistance and consistent hub sizes.

Be sure you have the right drainage catheter for each procedure, including your most challenging and complex cases. The choice is yours.


Contents

Percutaneous Access	5
Aprima™ Access Nonvascular Introducer Set.....	6-7
Jeffrey Wire Guide Exchange Set.....	8
Multipurpose and Abscess Drainage	11
Dawson-Mueller Multipurpose Drainage Catheter	12-13
Multipurpose Drainage Catheter	14
Universal Curved Drainage Catheter.....	15
Gordon Large-Bore Drainage Catheter	16
Thal-Quick Abscess Drainage Set	17
Yueh Centesis Catheter Needle	18
Intro-Tip™ Pneumothorax Set	19
Biliary Drainage	21
Biliary Drainage Catheter	22-23
Wittich Nitinol Stone Basket	24
Transluminal Biliary Biopsy Forceps Set	25
Uroradiology Drainage	27
Amplatz Ureteral Stent Set.....	28-29
Cope Nephroureterostomy Stent.....	30
Accessories	33
Low-Pressure Large Bore Three-Way Stopcock	34
Hydrophilic Dilator	34
Drainage Pouch Set.....	35
Connecting Tube with Drainage Bag Connector.....	35
Wire Guides	37
Coons Interventional Wire Guide	38
Lunderquist® Extra-Stiff Support Wire Guide	38
Amplatz Support Wire Guide	39



Percutaneous Access


Aprima™ Access Nonvascular Introducer Set.....	6-7
Jeffrey Wire Guide Exchange Set.....	8

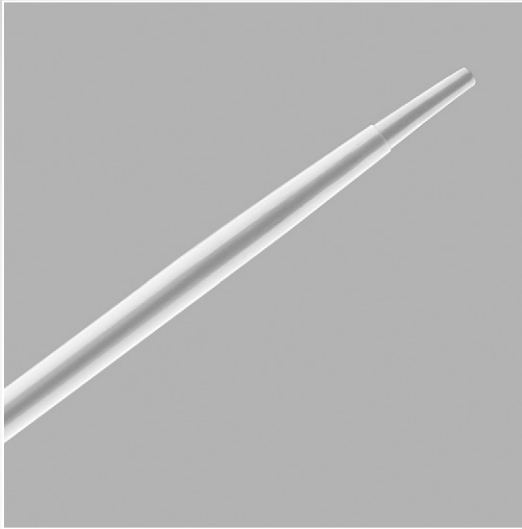

Aprima™ Access Nonvascular Introducer Set


Used for single-puncture percutaneous access to facilitate placement of a working wire guide for drainage procedures.

- The Transitionless-Tip™ design requires less insertion force and virtually eliminates hang-ups during entry.
- The entire shaft and distal tip are radiopaque to maximize visibility during placement.
- The set includes an EchoTip® echogenic needle, a Cope mandril wire guide, and a hydrophilic-coated coaxial introducer sheath, which work together to ease placement.
- The outer sheath permits the advancement of a .038 inch diameter wire guide while the .018 inch diameter wire guide remains in place as a safety wire.

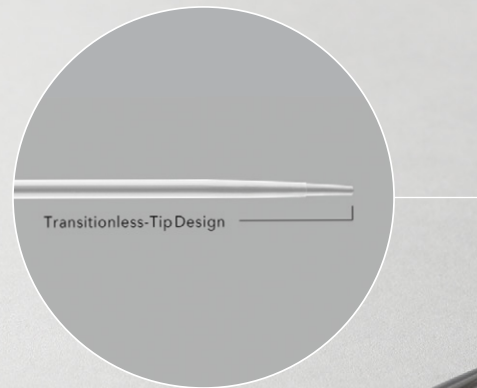

APRIMA ACCESS NONVASCULAR INTRODUCER SET

Components

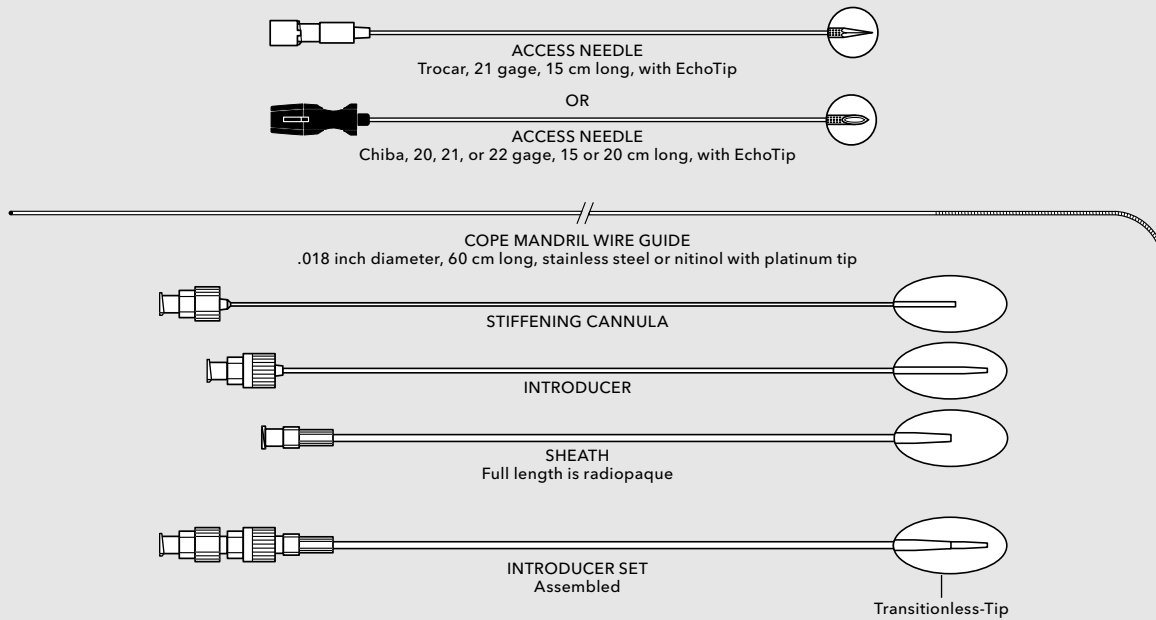


Order Number	Reference Part Number	Sheath ID/OD Fr	Needle gage/Length cm	Needle Tip Configuration	Wire Guide Type
G34253	NSSW-4.0-18-NPAS-100-HC-SST*	4.0/6.0	-	-	-
G52962	NPAS-100-HC-U-SST	4.0/6.0	22/15	Chiba	stainless steel
G52963	NPAS-100-HC-NT-U-SST	4.0/6.0	22/15	Chiba	nitinol
G52964	NPAS-104-HC-U-SST	4.0/6.0	21/15	trocár	stainless steel
G52965	NPAS-104-HC-NT-U-SST	4.0/6.0	21/15	trocár	nitinol
G52966	NPAS-104-HC-NT-JWG-U-SST**	4.0/6.0	21/15	trocár	nitinol
G52967	NPAS-105-HC-NT-U-SST	4.0/6.0	22/20	Chiba	nitinol
G25516	NPAS-108-HC-NT-U-SST	4.0/6.0	21/15	Chiba	nitinol
G25517	NPAS-120-HC-NT-U-SST	4.0/6.0	20/15	Chiba	nitinol

*This set does not include a needle or a wire guide.

**This set also includes a .038 inch, 100 cm heavy-duty wire guide with a curved tip.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

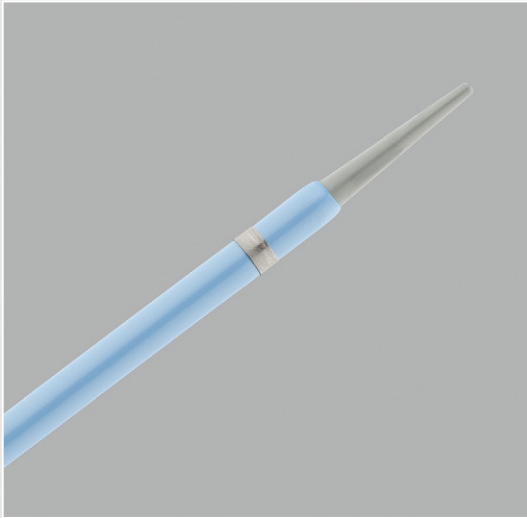


Jeffrey Wire Guide Exchange Set

Used for single-puncture percutaneous access and placement of two .038 inch diameter wire guides (not included) when indicated for interventional radiology procedures.

- A radiopaque band on the tip of the sheath identifies the location of the sheath's distal tip for accurate positioning.
- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during insertion.

Components

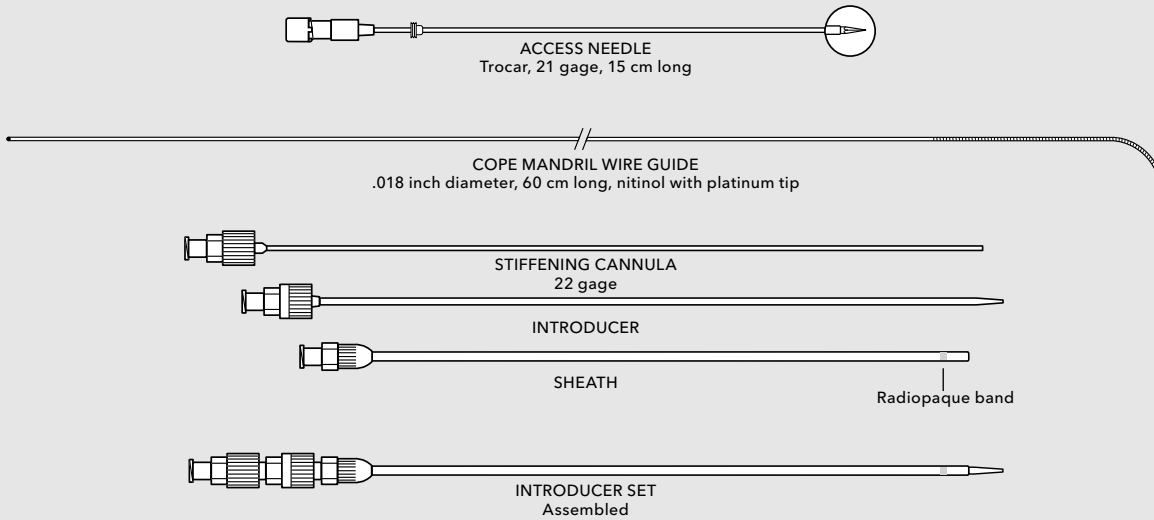

Order Number	Reference Part Number	Sheath ID/OD Fr	Needle gage/Length cm	Wire Guide Type
G12016	JWGE-104-RH-NT	6.5/8	21/15	nitinol

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Notes



Multipurpose and Abscess Drainage

Dawson-Mueller Multipurpose Drainage Catheter	12-13
Multipurpose Drainage Catheter.....	14
Universal Curved Drainage Catheter	15
Gordon Large-Bore Drainage Catheter.....	16
Thal-Quick Abscess Drainage Set.....	17
Yueh Centesis Catheter Needle	18
Intro-Tip™ Pneumothorax Set	19



Dawson-Mueller Multipurpose Drainage Catheter


Used for percutaneous drainage in a variety of drainage applications (e.g., nephrostomy, biliary, and abscess), either by direct stick or Seldinger access technique.

- The Intro-Tip™ design helps prevent the catheter from accordioning if it meets resistance during insertion.
- The easy-to-use, low-profile Mac-Loc® firmly fixates the 10 mm pigtail while maintaining patient comfort.
- Ultrathane® material enhances patient comfort and allows the catheter to be used safely with ethyl alcohol.
- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during catheter insertion.
- The 10 mm pigtail facilitates drainage in small cavities.



DAWSON-MUELLER MULTIPURPOSE DRAINAGE CATHETER

Components

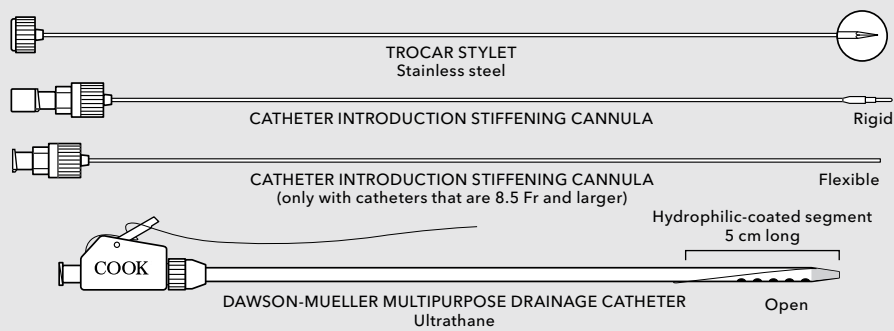

A securement device is included with catheters that are 6.3 Fr and larger

Order Number	Reference Part Number	Catheter Fr	Catheter Length cm	Catheter Sideports	Accepts Wire Guide Diameter inch	Cannula gage
G11018	ULT5.0-18-25-P-5S-CLDM-HC	5	25	5	.018	23 UTW*
G51594	ULT6.3-35-15-P-5S-CLDM-HC	6.3	15	5	.035	19 UTW*
G51595	ULT6.3-35-25-P-5S-CLDM-HC	6.3	25	5	.035	19 UTW*
G11020	ULT7.0-35-25-P-5S-CLDM-HC	7	25	5	.035	19 UTW*
G11386	ULT8.5-38-15-P-5S-CLDM-HC	8.5	15	5	.038	18
G09705	ULT8.5-38-25-P-5S-CLDM-HC	8.5	25	5	.038	18
G11387	ULT10.2-38-15-P-5S-CLDM-HC	10.2	15	5	.038	18
G09706	ULT10.2-38-25-P-5S-CLDM-HC	10.2	25	5	.038	18
G11720	ULT12.0-38-15-P-5S-CLDM-HC	12	15	5	.038	18
G10435	ULT12.0-38-25-P-5S-CLDM-HC	12	25	5	.038	18
G10436	ULT14.0-38-25-P-5S-CLDM-HC	14	25	5	.038	18

*Ultrathin wall

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Multipurpose Drainage Catheter

Used for percutaneous drainage in a variety of drainage applications (e.g., nephrostomy, biliary, and abscess), either by direct stick or Seldinger access technique.

- The easy-to-use, low-profile Mac-Loc® firmly fixates the 25 mm pigtail while maintaining patient comfort. Nonlocking options are available to suit physician preference.
- Ultrathane® material enhances patient comfort and allows the catheter to be used safely with ethyl alcohol.
- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during catheter insertion.
- The radiopaque band identifies the proximal area of the locking loop.

Components

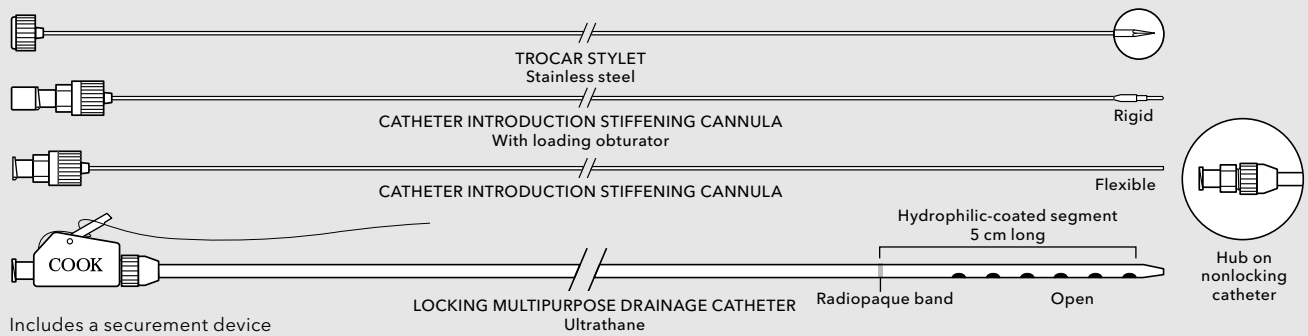

Order Number	Reference Part Number	Catheter Fr	Catheter Length cm	Catheter Sideports	Accepts Wire Guide Diameter inch	Trocar Stylet Needle gage
Mac-Loc Locking Multipurpose Drainage Catheter						
G09501	ULT8.5-38-25-P-6S-CLM-RH	8.5	25	6	.038	18
G09765	ULT8.5-38-45-P-6S-CLM-RH	8.5	45	6	.038	18
G09502	ULT10.2-38-25-P-6S-CLM-RH	10.2	25	6	.038	18
G09766	ULT10.2-38-45-P-6S-CLM-RH	10.2	45	6	.038	18
G28916	ULT10.2-38-60-P-6S-CLM-RH	10.2	60	6	.038	18
G09503	ULT12.0-38-25-P-6S-CLM-RH	12	25	6	.038	18
G09767	ULT12.0-38-45-P-6S-CLM-RH	12	45	6	.038	18
G09641	ULT12.0-38-60-P-6S-CLM-RH	12	60	6	.038	18
G09504	ULT14.0-38-25-P-6S-CLM-RH	14**	25	6	.038	18
G09768	ULT14.0-38-45-P-6S-CLM-RH	14**	45	6	.038	18
G28936	ULT14.0-38-60-P-6S-CLM-RH	14**	60	6	.038	18
Nonlocking Multipurpose Drainage Catheter*						
G06483	ULT8.5-38-25-P-6S-MDC	8.5	25	6	.038	18
G06484	ULT10.2-38-25-P-6S-MDC	10.2	25	6	.038	18
G06485	ULT12.0-38-25-P-6S-MDC	12	25	6	.038	18
G06482	ULT14.0-38-25-P-6S-MDC	14**	25	6	.038	18

*Does not include hydrophilic coating or radiopaque band.

**Please contact your local Cook representative for drainage catheters that are larger than 14 Fr.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Universal Curved Drainage Catheter

Used for percutaneous drainage in a variety of drainage applications (e.g., nephrostomy, biliary, and abscess), either by direct stick or Seldinger access technique.

- The Intro-Tip™ design helps prevent the catheter from accordioning if it meets resistance during insertion.
- Ultrathane® material enhances patient comfort and allows the catheter to be used safely with ethyl alcohol.
- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during catheter insertion.

Components

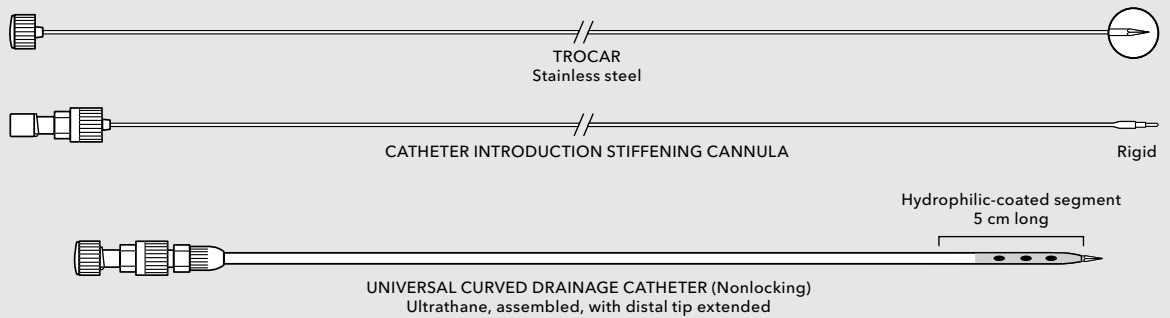

Includes a securement device

Order Number	Reference Part Number	Catheter Fr	Catheter Length cm	Catheter Sideports	Accepts Wire Guide Diameter inch	Trocar Stylet Needle gage
G10956	ULT6.0-35-25-P-3S-UCD-HC	6	25	3	.035	19 UTW*
G11340	ULT7.0-38-15-P-3S-UCD-HC	7	15	3	.038	18
G10957	ULT7.0-38-25-P-3S-UCD-HC	7	25	3	.038	18
G10958	ULT8.5-38-25-P-3S-UCD-HC	8.5	25	3	.038	18
G10959	ULT10.2-38-25-P-3S-UCD-HC	10.2	25	3	.038	18

*Ultrathin wall

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Gordon Large-Bore Drainage Catheter


Used for abscess drainage.

- The large-bore catheter facilitates drainage of thick fluids that will not adequately drain through smaller devices.
- The catheter can be inserted via a direct stick with the 18 gage trocar stylet that is included in the set, or by using the Seldinger (over-the-wire) technique.
- Ultrathane® material enhances patient comfort and allows the catheter to be used safely with ethyl alcohol.
- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during catheter insertion.

Standard catheter components

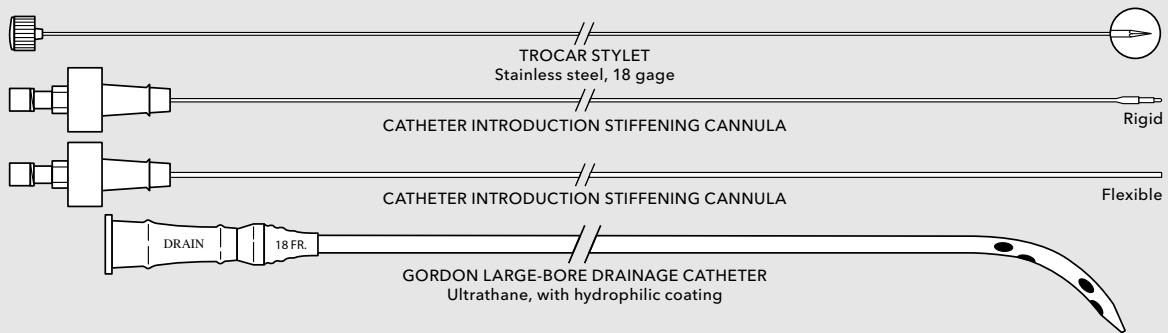

Coaxial catheter components

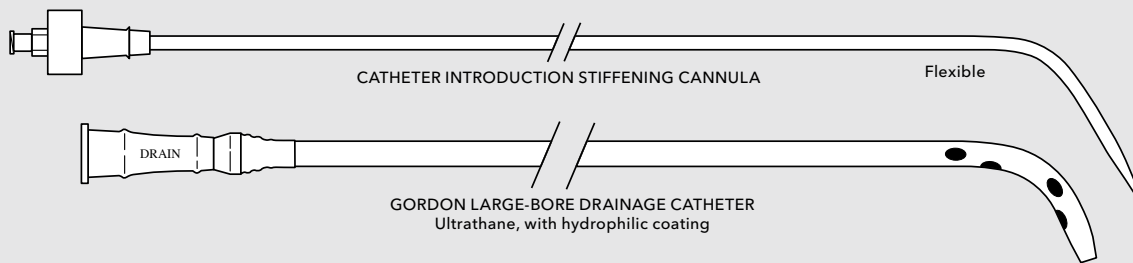

Order Number	Reference Part Number	Catheter Fr	Catheter Length cm	Catheter Sideports	Accepts Wire Guide Diameter inch
Standard Catheter					
G12509	ULT16.0-38-40-P-6S-MPA-GLD-HC	16	40	6	.038
G12510	ULT18.0-38-40-P-6S-MPA-GLD-HC	18	40	6	.038
G12511	ULT20.0-38-40-P-6S-MPA-GLD-HC	20	40	6	.038
Coaxial Catheter					
G12512	ULT22.0-38-40-P-6S-MPA-GLD-HC	22	40	6	.038

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Thal-Quick Abscess Drainage Set

Used for abscess drainage.

- The large-bore, clear polyvinylchloride catheter with a radiopaque stripe allows fluids to be seen during the drainage procedure.
- The EchoTip® needle cannula's etched tip enhances visualization under ultrasound.
- The heavy-duty wire guide offers increased support to facilitate placement of the catheter.
- Multiple large, oval sideports increase drainage capability.

Components

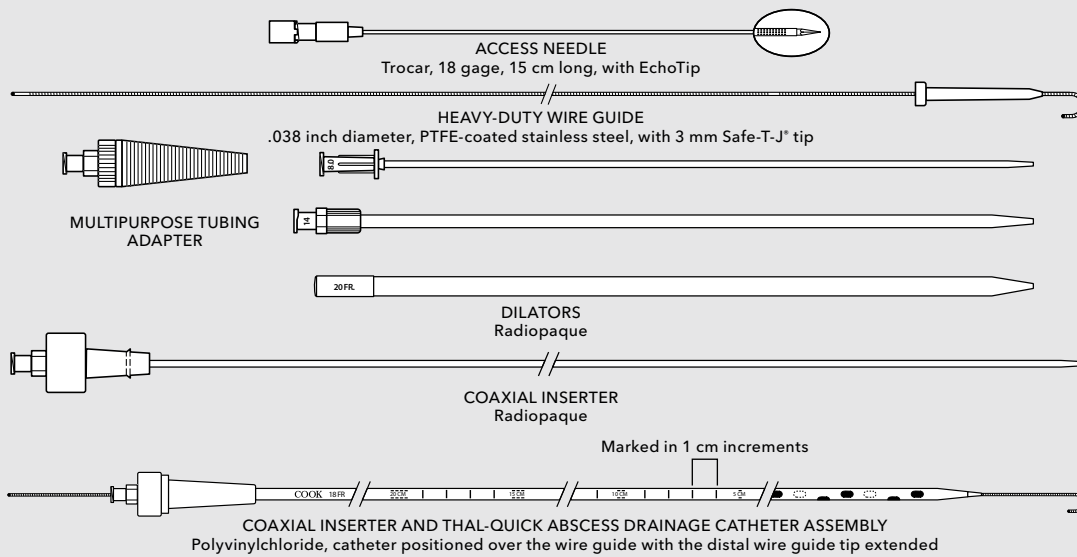

Order Number	Reference Part Number	Catheter Fr	Catheter Length cm	Catheter Sideports
G06336	TQAS-1200-J	12	33	9
G08454	TQAS-1400-J	14	41	9
G06339	TQAS-1600-J	16	41	9
G06343	TQAS-1800-J	18	41	9
G06345	TQAS-2000-J	20	41	9
G06348	TQAS-2400-J	24	41	9
G06354	TQAS-2800-J	28	41	9

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Yueh Centesis Catheter Needle


Used for percutaneous fluid aspiration.

- The hollow needle cannula design permits aspiration of fluid prior to removing the needle cannula.
- Four sideports located on the distal tip of the catheter facilitate drainage within small cavities.
- A pigtail loop option and echogenic options are available to suit physician preference.

Straight catheter needle components

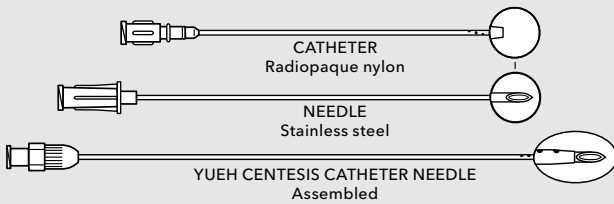

Pigtail catheter needle components

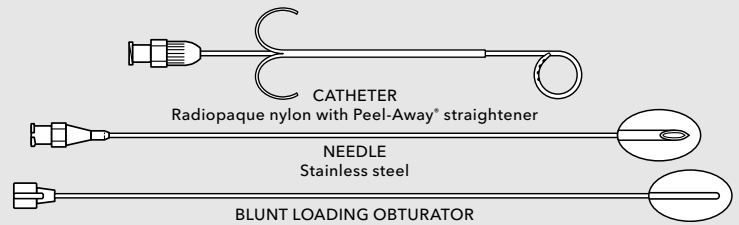

Order Number	Reference Part Number	Catheter Fr	Catheter Length cm	Catheter Sideports	Needle gage
Straight Catheter					
G09488	DTVN-4.0-20XT-7.0-YUEH	4	7	4	20 XTW*
G09646	DTVN-4.0-20XT-10.0-YUEH	4	10	4	20 XTW*
G09489	DTVN-5.0-19-7.0-YUEH	5	7	4	19
G09490	DTVN-5.0-19-10.0-YUEH	5	10	4	19
G09492	DTVN-5.0-19-15.0-YUEH	5	15	4	19
G09491	DTVN-5.0-19-20.0-YUEH	5	20	4	19
G12996	DTVN-6.0-19UT-10.0-YUEH	6	10	4	19 UTW**
G12892	DTVN-8.0-16-7.0-YUEH	8	7	4	16
Straight Catheter – Echogenic					
G56317	DTVN-4.0-20XT-7.0-YUEH-U	4	7	4	20 XTW*
G55528	DTVN-5.0-19-7.0-YUEH-U	5	7	4	19
Pigtail Catheter					
G10100	DTVN-5.0-19-15.0-YUEH-RDC-070896	5	15	4	19

*Extra-thin wall

**Ultrathin wall

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Intro-Tip™ Pneumothorax Set

Used to treat simple spontaneous pneumothorax and tension pneumothorax.

- The catheter is introduced with a one-stick trocar technique.
- The Intro-Tip facilitates placement of the catheter and helps prevent the catheter from accordioning back over the trocar needle assembly when it encounters resistance during insertion.
- The hydrophilic coating on the distal 10 cm of the catheter, when activated, becomes very lubricious, reducing friction during catheter insertion.
- The connecting tube can be connected to a suction apparatus or to the Heimlich chest drain valve that is included in the set.

Components

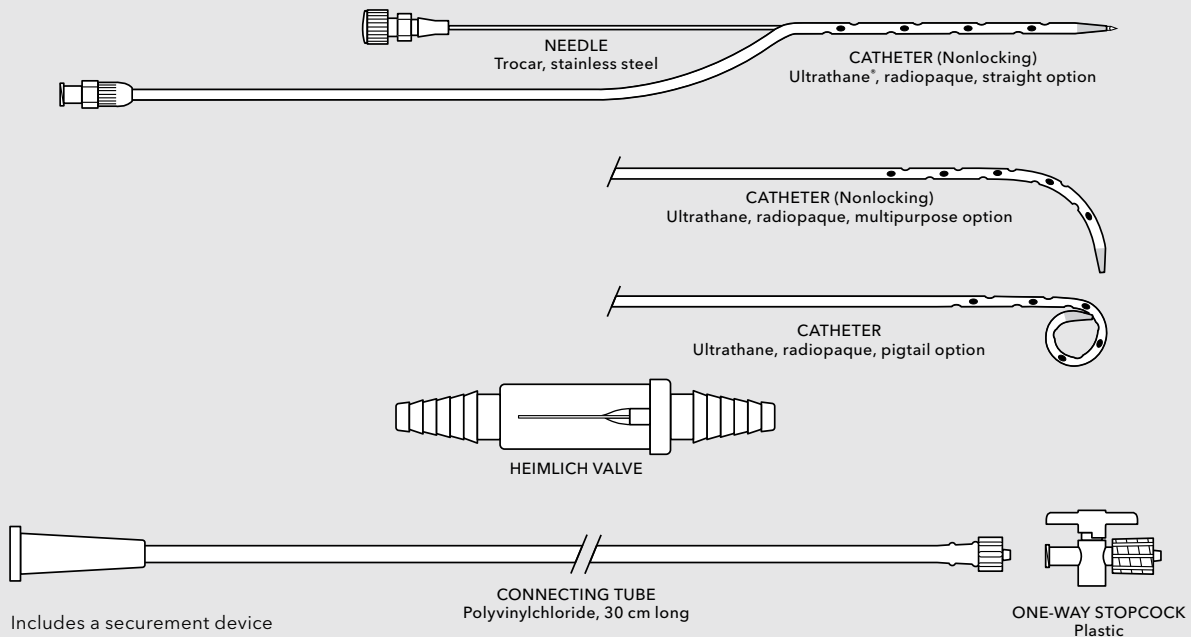

Order Number	Reference Part Number	Catheter Fr/Length cm	Catheter Sideports	Catheter Tip Configuration	Needle gage/Length cm	Needle Tip
G09763	UTPT-1020-0-RIV-HC	10.2/30	18	straight	18/20	trocar
G09800	UTPT-1020-MPA-RIV-HC	10.2/30	18	multipurpose	18/20	trocar
G09764	UTPT-1020-PIG-RIV-HC	10.2/30	18	pigtail	18/20	trocar

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



Biliary Drainage

Biliary Drainage Catheter	22-23
Wittich Nitinol Stone Basket	24
Transluminal Biliary Biopsy Forceps Set	25



Biliary Drainage Catheter


Used for biliary drainage applications.

- The easy-to-use, low-profile Mac-Loc® mechanism firmly fixates the loop while maintaining patient comfort.
- Ultrathane® material enhances patient comfort and allows the catheter to be used safely with ethyl alcohol.
- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during catheter insertion.
- The radiopaque band marks the location of the most proximal sideport.




BILIARY DRAINAGE CATHETER

Biliary drainage catheter components

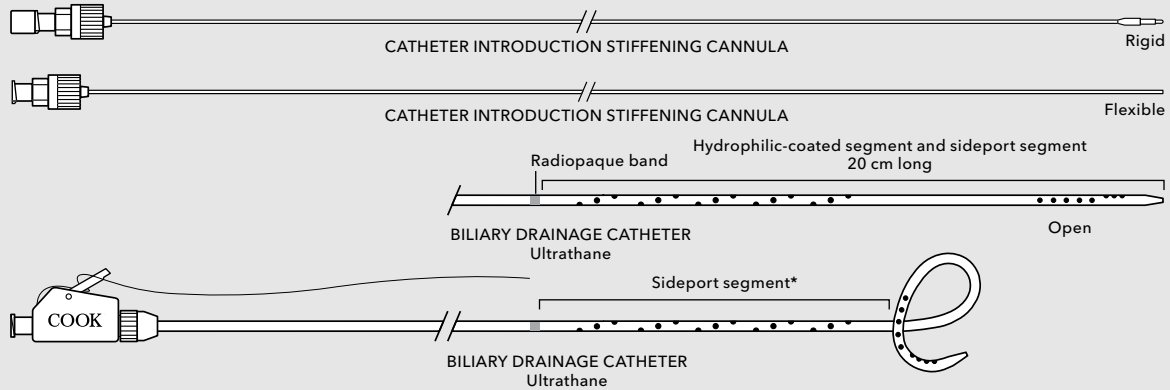

Includes a securement device

Multipurpose drainage catheter components

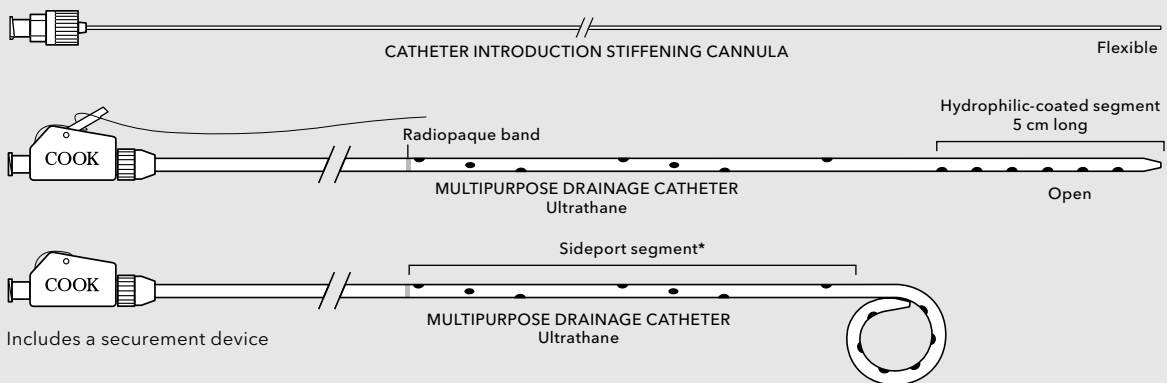

Includes a securement device

Order Number	Reference Part Number	Catheter Fr	Catheter Length cm	Catheter Sideports	Sideport Segment Length cm	Accepts Wire Guide Diameter inch
Biliary Drainage Catheter						
G09497	ULT8.5-38-40-P-32S-CLB-RH	8.5	40	32	8*	.038
G09498	ULT10.2-38-40-P-32S-CLB-RH	10.2	40	32	8*	.038
G09499	ULT12.0-38-40-P-32S-CLB-RH	12	40	32	8*	.038
G09500	ULT14.0-38-40-P-32S-CLB-RH	14**	40	32	8*	.038
Biliary Drainage Catheter – Extended Length Sideports						
G13359	ULT8.5-38-40-P-CLBS-26-RH	8.5	40	-	16*	.038
G13353	ULT10.2-38-40-P-CLBS-26-RH	10.2	40	-	16*	.038
G13355	ULT12.0-38-40-P-CLBS-26-RH	12	40	-	16*	.038
G13354	ULT10.2-38-40-P-CLBS-28-RH	10.2	40	-	18*	.038
Multipurpose Drainage Catheter – Extended Length Sideports						
G27048	ULT8.5-38-35-P-CLMS-16-RH-FS	8.5	35	-	16*	.038
G27043	ULT10.2-38-35-P-CLMS-16-RH-FS	10.2	35	-	16*	.038
G27045	ULT12.0-38-35-P-CLMS-16-RH-FS	12	35	-	16*	.038

*Sideport segment length is measured from the radiopaque band to the proximal formation of the locking loop.

**Please contact your local Cook representative for drainage catheters that are larger than 14 Fr.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Wittich Nitinol Stone Basket

Used to remove stones or stone fragments from the gallbladder, biliary tract, renal pelvis, and ureter.

- The kink-resistance and memory characteristics of the 6-wire nitinol basket allow it to retain its shape after repeated twisting and flexing. The precurved, angled tip of the introducer sheath provides directional control.
- The introducer's smooth taper from sheath to dilator eases introduction.
- The nitinol bulb tip is atraumatic to anatomic structures.
- The shape memory and flexibility of the nitinol basket facilitate access to and entrapment of stones.
- The highly radiopaque tip enhances visibility of the introducer sheath's position during manipulation.

Components

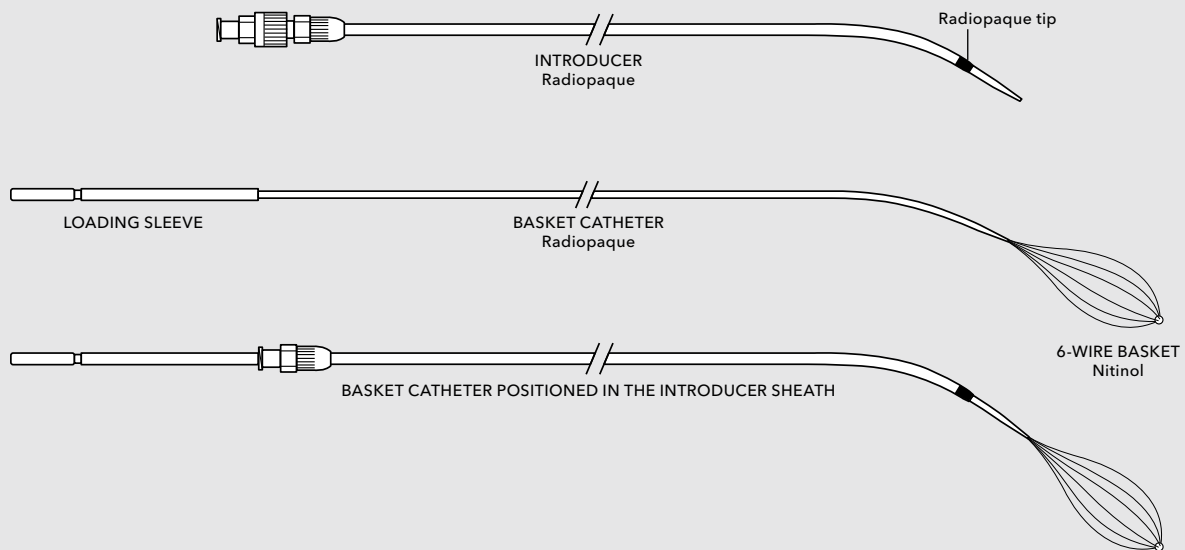

Order Number	Reference Part Number	Introducer Diameter Fr	Introducer Length cm	Basket Diameter cm	Basket Length cm
G07313	WNSB-8.5-50	8.5	50	1.8	2.5
G05877	WNSB-12-24	12	24	2.2	4.5

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Transluminal Biliary Biopsy Forceps Set

Used to establish percutaneous access for drainage procedures and to perform transluminal forceps biopsies of biliary strictures in a single, efficient procedure.

- The Flexor® sheath's position against the stricture allows the forceps to penetrate more deeply and reliably obtain an accurate sample.
- The forceps are designed so that users can operate them with one hand while maintaining control of the Flexor sheath.
- The set includes all of the components needed to access the biliary tree for transluminal biopsy.

Components

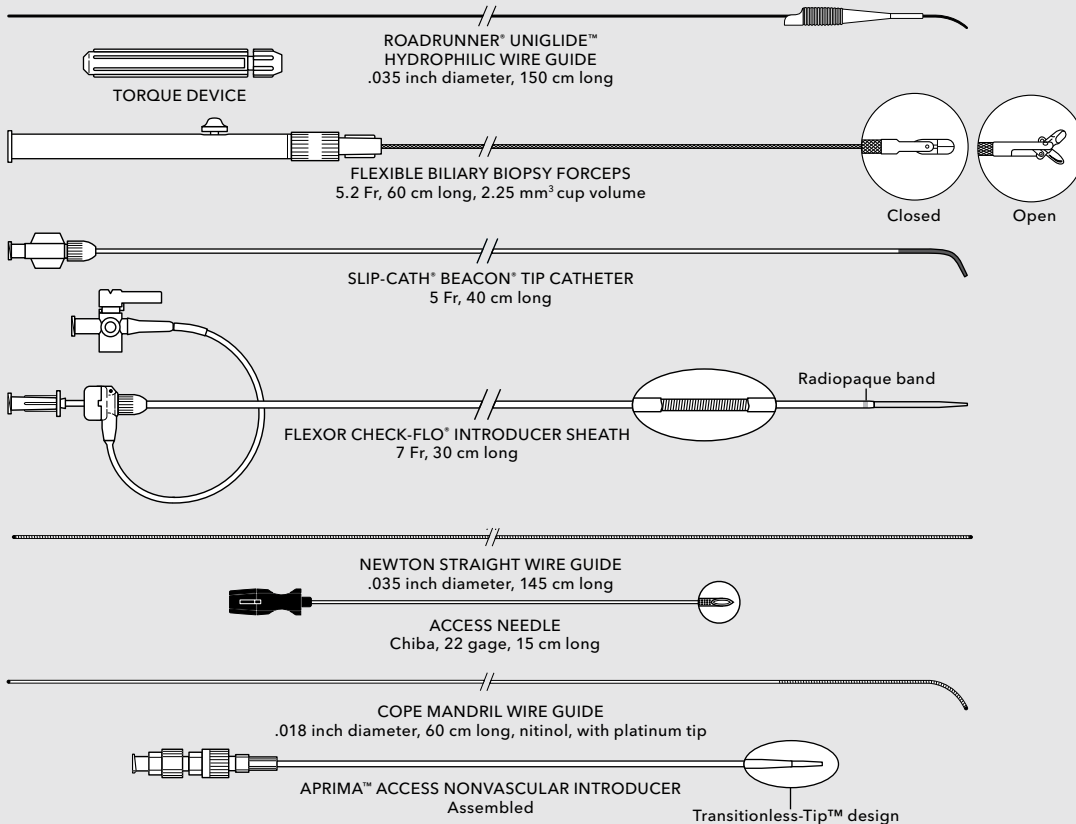


Order Number	Reference Part Number	Introducer Fr	Introducer Length cm	Wire Guide Diameter inch	Wire Guide Length cm	Biopsy Cup Volume mm ³
G26088	BBFS-100	7	30	.035	145/150	2.25

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



Uroradiology Drainage

Amplatz Ureteral Stent Set.....	28-29
Cope Nephroureterostomy Stent.....	30

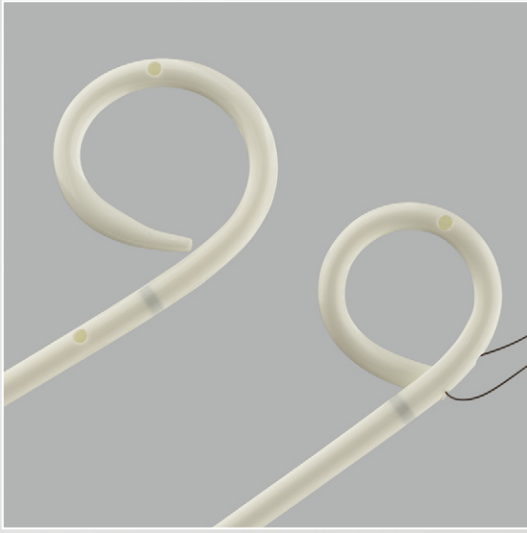

Amplatz Ureteral Stent Set


Used for internal drainage of urine.

- The radiopaque bands, located just proximal to the pigtails, help to accurately position the stent.
- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during stent insertion.
- Ultrathane® material enhances patient comfort and allows the device to be used safely with ethyl alcohol.




AMPLATZ URETERAL STENT SET

Components

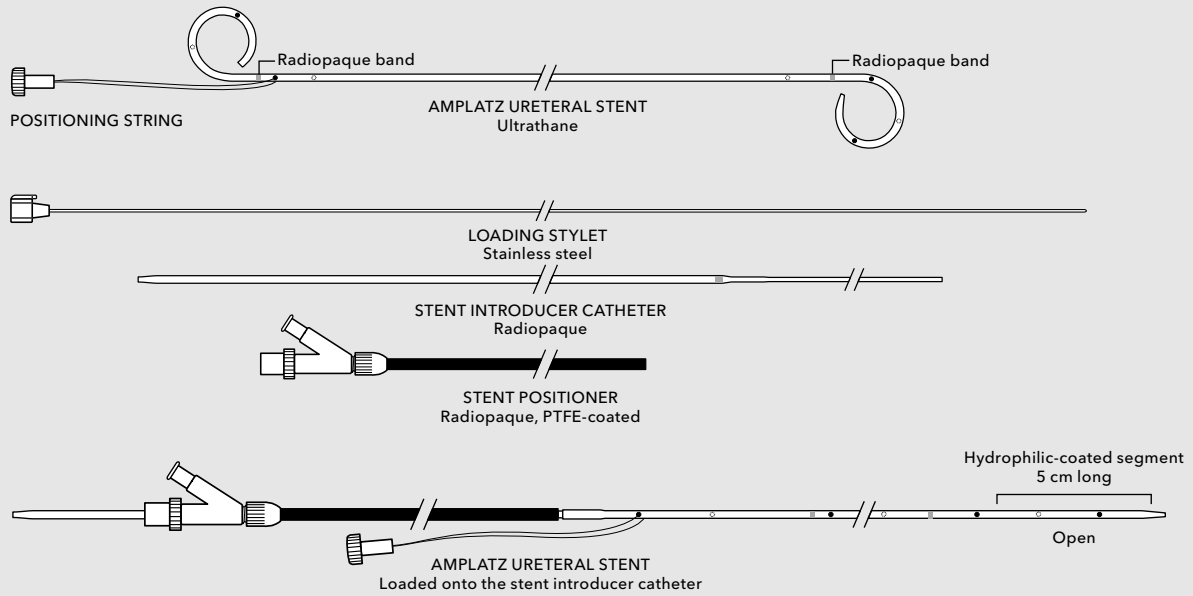

Order Number	Reference Part Number	Stent Fr	Stent Length cm	Stent Sideports	Accepts Wire Guide Diameter inch
G09977	UTSSW-8.5-10-AMP-RH	8.5	10	6	.035
G09989	UTSSW-8.5-12-AMP-RH	8.5	12	6	.035
G11255	UTSSW-8.5-16-AMP-RH	8.5	16	6	.035
G09865	UTSSW-8.5-20-AMP-RH	8.5	20	6	.035
G09708	UTSSW-8.5-22-AMP-RH	8.5	22	6	.035
G09709	UTSSW-8.5-24-AMP-RH	8.5	24	6	.035
G09710	UTSSW-8.5-26-AMP-RH	8.5	26	6	.035
G09711	UTSSW-8.5-28-AMP-RH	8.5	28	6	.035
G12049	UTSSW-10.2-16-AMP-RH	10.2	16	6	.038
G11367	UTSSW-10.2-20-AMP-RH	10.2	20	6	.038
G09712	UTSSW-10.2-22-AMP-RH	10.2	22	6	.038
G09713	UTSSW-10.2-24-AMP-RH	10.2	24	6	.038
G09714	UTSSW-10.2-26-AMP-RH	10.2	26	6	.038
G09715	UTSSW-10.2-28-AMP-RH	10.2	28	6	.038
G10054	UTSSW-10.2-30-AMP-RH	10.2	30	6	.038

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

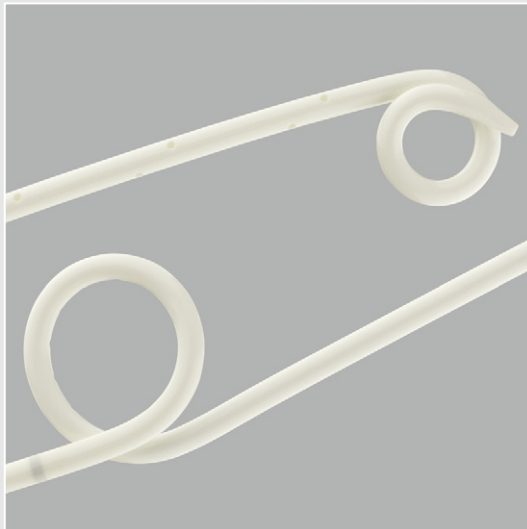


Cope Nephroureterostomy Stent

Used for internal drainage from the ureteropelvic junction to the bladder while maintaining external access to the stent.

- The easy-to-use, low-profile Mac-Loc® locking mechanism firmly fixates the proximal pigtail within the kidney while maintaining patient comfort.
- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during stent insertion.
- The proximal radiopaque band offers enhanced visibility for accurate positioning in the kidneys.
- Ultrathane® material enhances patient comfort and allows the device to be used safely with ethyl alcohol.

Components

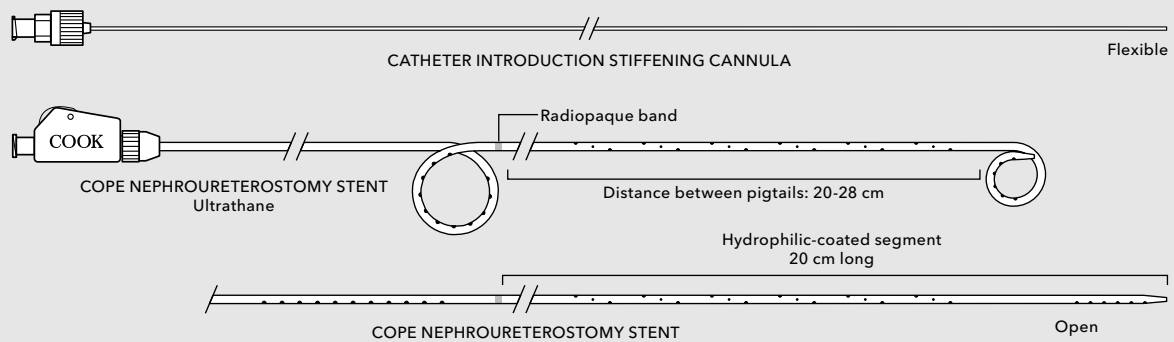

Includes a securement device

Order Number	Reference Part Number	Catheter Fr	Accepts Wire Guide Diameter inch	Distance Between Pigtails cm
Sideports in the Pigtails Only				
G48170	ULT8.5-8.5-20-NUCL-A-RH	8.5	.038	20
Sideports in the Pigtails and the Entire Distal Segment				
G48173	ULT8.5-8.5-22-NUCL-B-RH	8.5	.038	22
G48176	ULT8.5-8.5-24-NUCL-B-RH	8.5	.038	24
G48180	ULT8.5-8.5-26-NUCL-B-RH	8.5	.038	26
G48183	ULT8.5-8.5-28-NUCL-B-RH	8.5	.038	28
G48160	ULT10.2-10.2-20-NUCL-B-RH	10.2	.038	20
G48162	ULT10.2-10.2-22-NUCL-B-RH	10.2	.038	22
G48164	ULT10.2-10.2-24-NUCL-B-RH	10.2	.038	24
G48166	ULT10.2-10.2-26-NUCL-B-RH	10.2	.038	26
G48168	ULT10.2-10.2-28-NUCL-B-RH	10.2	.038	28

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Notes



Accessories

Low-Pressure Large Bore Three-Way Stopcock.....	34
Hydrophilic Dilator	34
Drainage Pouch Set	35
Connecting Tube with Drainage Bag Connector	35



Low-Pressure Large Bore Three-Way Stopcock

Used for larger drainage catheter connections to facilitate drainage.

- Features two female Luer locks, and one male Luer lock.
- Arrows indicate the direction of the fluid pathway, facilitating safe and convenient disposal of fluids.
- The large inner diameter accommodates larger drains without limiting drainage flow.

Order Number

Reference Part Number

G07840

HF-PTWSC-2FLL-MLL-R-LP

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



Hydrophilic Dilator

Used for dilating puncture sites or catheter tracts.

- The hydrophilic coating, when activated, becomes very lubricious, reducing friction during dilator insertion.

Order Number	Reference Part Number	Dilator Fr	Length cm	Accepts Wire Guide Diameter inch
G10205	JCD4.0-25-20-HC	4	20	.025
G10282	JCD5.0-35-20-HC	5	20	.035
G10283	JCD5.0-38-20-HC	5	20	.038
G10284	JCD6.0-35-20-HC	6	20	.035
G10285	JCD6.0-38-20-HC	6	20	.038
G10208	JCD7.0-38-20-HC	7	20	.038
G10289	JCD8.0-38-20-HC	8	20	.038
G10290	JCD9.0-38-20-HC	9	20	.038
G10207	JCD10.0-38-20-HC	10	20	.038
G10292	JCD12.0-38-20-HC	12	20	.038
G10293	JCD14.0-38-20-HC	14	20	.038
G10206	JCD16.0-38-20-HC	16	20	.038

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



Drainage Pouch Set

Used for collection of nephrostomy and biliary drainage.

- The pouch is worn around the waist and is held in place by a soft elastic belt.
- A reed valve mechanism prevents reflux of fluid.

Order Number	Reference Part Number	Volume mL	Connector Type
Drainage Pouch Set – Sterile			
G14653	515003	300	single
G14654	515004	300	double
G15133	515011	500	single
G15134	515012	500	double
Drainage Pouch Set – Nonsterile			
G14516	505003	300	single
G14517	505004	300	double
G15136	505011	500	single
G15137	505012	500	double
Drainage Pouch Only – Sterile			
G15918	515000	300	-

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



Connecting Tube with Drainage Bag Connector

Used to connect an external drainage catheter to a drainage pouch.

- The one-way stopcock controls drainage flow during drainage-pouch exchange.

Order Number	Reference Part Number	Tube Fr	Length cm	Connector Type
G04711	CTU14.0-10-ST	14	10	stopcock
G02791	CTU14.0-30	14	30	Luer lock
G02278	CTU14.0-30-ST	14	30	stopcock
G02327	CTU14.0-40-ST	14	40	stopcock

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



Wire Guides

Coons Interventional Wire Guide	38
Lunderquist® Support Wire Guide	38
Amplatz Support Wire Guide	39



Coons Interventional Wire Guide

Used for interventional biliary drainage, abscess drainage, and urology procedures.

Order Number	Reference Part Number	Diameter inch	Length cm	Flexible Tip Length cm
G12268	THSF-35-80-COONS	.035	80	15
G02356	THSF-35-145-COONS	.035	145	15
G04494	THSF-38-80-COONS	.038	80	15
G02462	THSF-38-100-COONS	.038	100	15
G02324	THSF-38-145-COONS	.038	145	15

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.




Lunderquist® Extra-Stiff Support Wire Guide

Used for complex diagnostic and interventional procedures.

Order Number	Reference Part Number	Diameter inch	Length cm	Flexible Tip Length cm
G49224	TSMG-35-90-4-LES	.035	90	4
G49225	TSMG-35-90-7-LES	.035	90	4

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

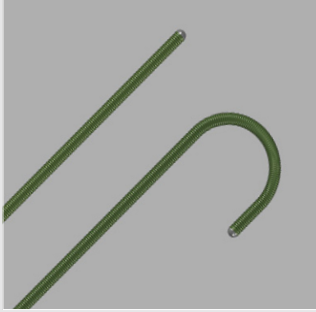

Amplatz Support Wire Guide

Used for access and catheterization procedures.

Order Number	Reference Part Number	Diameter inch	Length cm
Straight – Extra-Stiff PTFE-Coated Stainless Steel			
G04003	THSF-35-80-AES	.035	80
G03095	THSF-35-145-AES	.035	145
G03138	THSF-38-80-AES	.038	80
G03031	THSF-38-145-AES	.038	145
Curved – Extra-Stiff PTFE-Coated Stainless Steel			
G04237	THSCF-35-80-3-AES	.035	80
G03540	THSCF-38-100-3-AES	.038	100
G03047	THSCF-38-145-3-AES	.038	145
Straight – Ultra-Stiff PTFE-Coated Stainless Steel			
G10168	THSF-35-80-AUS	.035	80
G09954	THSF-35-145-AUS	.035	145
G10169	THSF-38-80-AUS	.038	80
G09957	THSF-38-145-AUS	.038	145

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Customer Service

EMEA: EDI - www.cookmedical.com/edi.do
Distributors: +353 61239240, ssc.distributors@cookmedical.com
Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001633, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
Finland: +358 972519996, fi.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com
Norway: +47 23162968, no.orders@cookmedical.com
Spain: +34 912702691, es.orders@cookmedical.com
Sweden: +46 858769468, se.orders@cookmedical.com
Switzerland - French: +41 448009609, fr.orders@cookmedical.com
Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
Switzerland - German: +41 448009609, de.orders@cookmedical.com
United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
E-mail: orders@cookmedical.com

Australia:
Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283
E-mail: cau.custserv@cookmedical.com

