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
**FirstStic™ N Introducer**

**Multi-language Instructions For Use (IFU)**

# TABLE OF CONTENTS

## Instructions for use:

1. English. . . . .	1
2. Bulgarian . . . . .	2
3. Czech . . . . .	3
4. Danish . . . . .	4
5. German . . . . .	6
6. Estonian . . . . .	7
7. Finnish. . . . .	8
8. French . . . . .	9
9. Croatian. . . . .	11
10. Greek. . . . .	12
11. Hungarian . . . . .	13
12. Italian. . . . .	15
13. Lithuanian. . . . .	16
14. Latvian . . . . .	17
15. Dutch. . . . .	19
16. Polish. . . . .	20
17. Portuguese . . . . .	21
18. Romanian . . . . .	23
19. Slovak. . . . .	24
20. Slovenian. . . . .	25
21. Spanish . . . . .	26
22. Swedish. . . . .	28
23. Turkish. . . . .	29

# ENGLISH

## FirstStic™ N Introducer

STERILE EO

### DEVICE DESCRIPTION

The FirstStic™ N Introducer is a notched introducer needle with a vented fitting and a splittable sheath introducer to facilitate percutaneous placement of a peripherally inserted central catheter (PICC) or midline catheter. The notched needle and clear sheath allow for a quick visual flashback when the vessel is penetrated. The device does not contain natural rubber latex and does not contain DEHP.

### INDICATION FOR USE

The catheter introducer is a sheath used to facilitate placing a catheter through the skin into a vein or artery.

### WARNINGS

- Reuse or reprocessing has not been evaluated and may lead to device failure and subsequent patient illness, infection and/or injury. DO NOT reuse, reprocess or re-sterilize this device.
- Do not use if package is damaged.
- Withdrawing the sheath rapidly might cause damage to the catheter. During withdrawal, slowly peel and remove the sheath.
- Do not attempt to re-insert a partially or completely withdrawn needle as it might cause damage to the catheter introducer.
- Do not withdraw the catheter back through the peelable sheath if it becomes impossible to advance the catheter into the right position. The sheath and catheter must be withdrawn simultaneously. Withdrawing the catheter through the cannula can cause a catheter embolism.
- Do not grasp the catheter tightly with forceps as it might damage the catheter.

### INSTRUCTION FOR USE

1. Prepare the insertion site per hospital procedures. Apply a tourniquet above the intended insertion site in a manner easily accessible for removal. Change to sterile gloves.
2. Remove protective cover from the catheter introducer while gripping the sheath wings.
3. Using a slow but firm motion, insert the catheter introducer into the vessel, ensuring the bevel is facing up with a 15 to 30 degree entry angle. Entry should be slow and at a shallow depth. Confirm entry into the vessel by observing blood flashback. Once entry is gained, lower the angle of the introducer needle and advance it inside the vessel.
4. Remove the tourniquet.
5. Withdraw the needle while holding the introducer sheath stationary.
6. Insert catheter through sheath opening using forceps without teeth. Use short, steady strokes and advance the catheter to the desired position.
7. Secure the catheter by applying light finger pressure beyond the sheath opening and slowly withdraw the sheath.
8. Gently bend the wings outward to crack apart and peel the sheath while maintaining forward pressure on the catheter. Be careful not to dislodge the catheter from the vessel.

## DISPOSAL

After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

## STORAGE

Store at controlled room temperature.

**CAUTION:** Federal Law (USA) restricts this device to sale by or on the order of a physician.

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**Argon Medical Devices, Inc.,**